



1. Location: There are three possible locations that will give a successful stimulation of the common peroneal nerve (dorsiflexion). The **A** position aligns with the fibula head, while **B** and **C** are aligned with a prominent lateral tendon.

The reasons for choosing an alternative fitting position include: skin sensitivity or lesions at the other position(s), better stimulation of the nerve because of anatomical variations, better comfort or ease of fitting, or the position of dressings.

2. Cleaning: Wash and dry the skin where the device will be fitted.

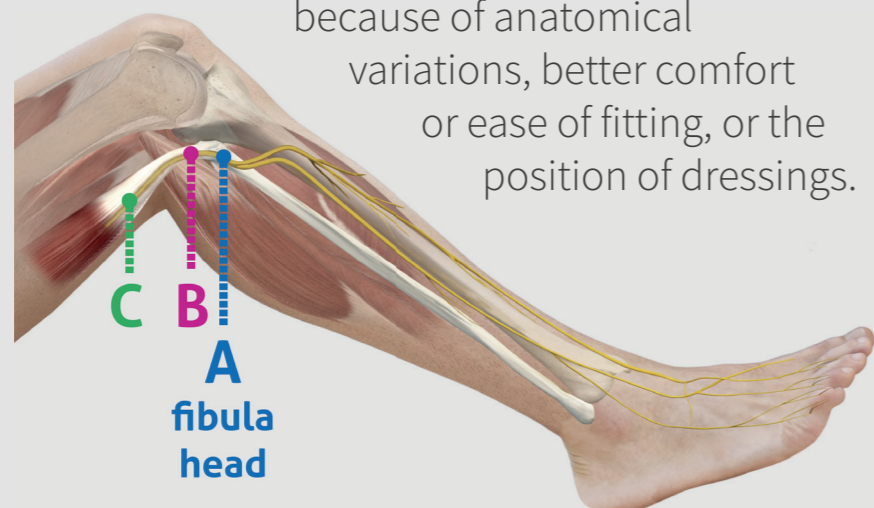

3^B. Fitting: Place the marker line ►►●◄◄ on the lateral/outer/biceps femoris tendon below the crease of the knee.

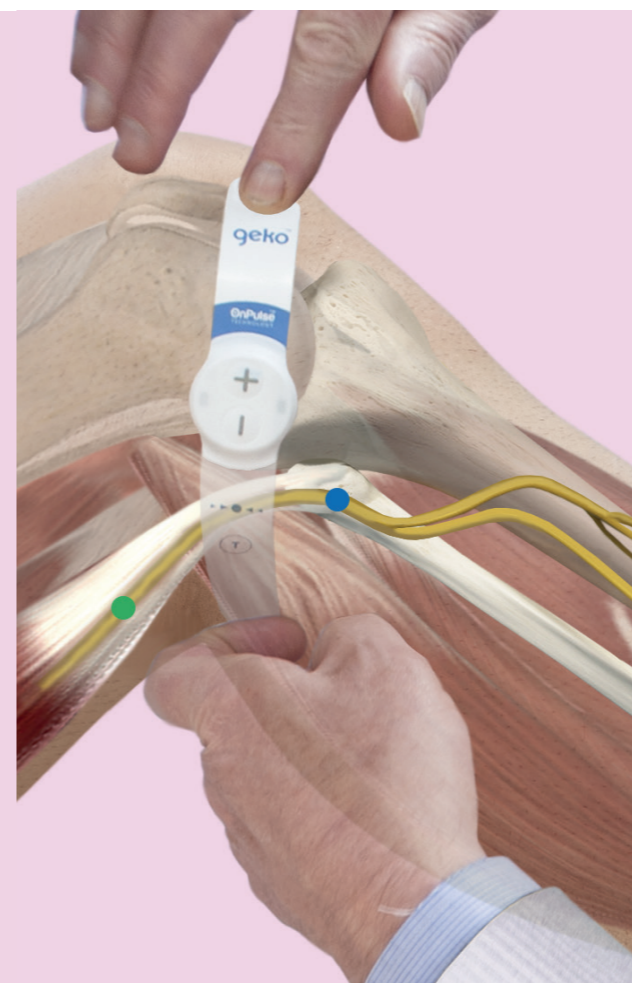


4. Turning On: To turn on, use a short press of the ⊕ button.



5. Settings: Use the ⊕ button to increase the setting and ⊖ button to decrease. Increase the setting until you get a rhythmic upwards and outwards movement of the foot.



3^A. Fitting: Remove the film from the geko™ device and place the marker line ►►●◄◄ over the fibula head.

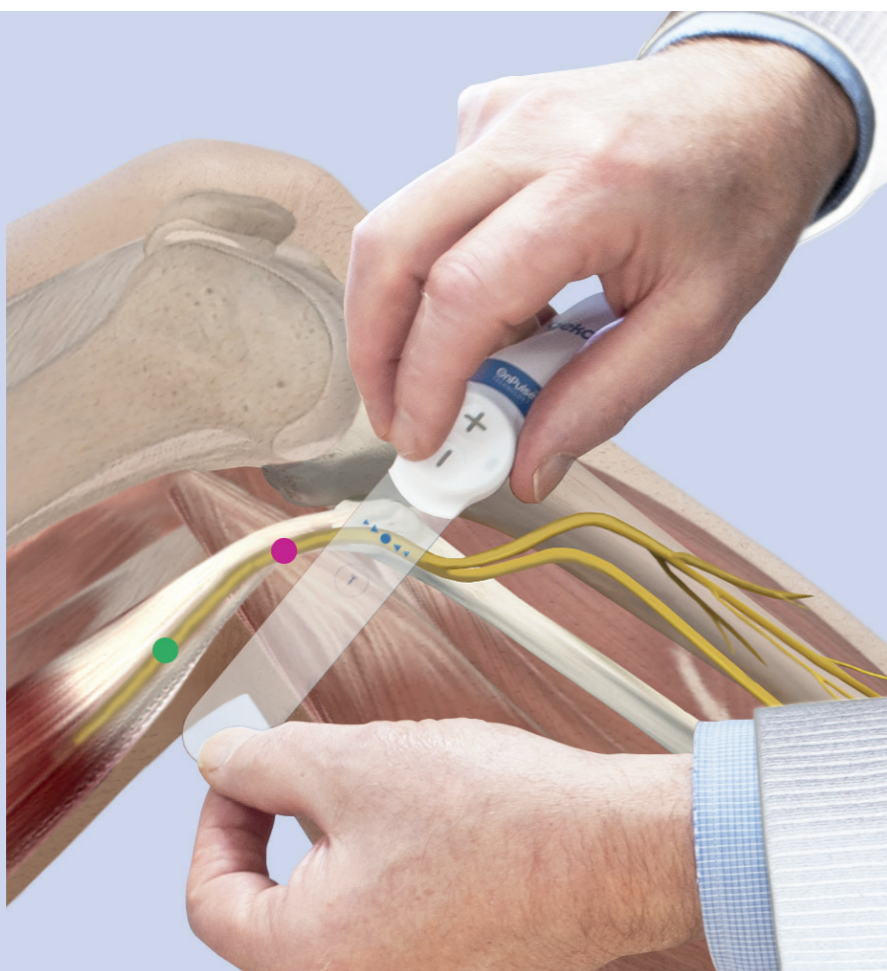


3^C. Fitting: Place the marker line ►►●◄◄ on the lateral/outer tendon just above the crease of the knee.



6. Switching Off: To turn it off, hold ⊖ button down for 3 seconds.



7. Removing: Remove carefully in one piece, to avoid damaging the skin.

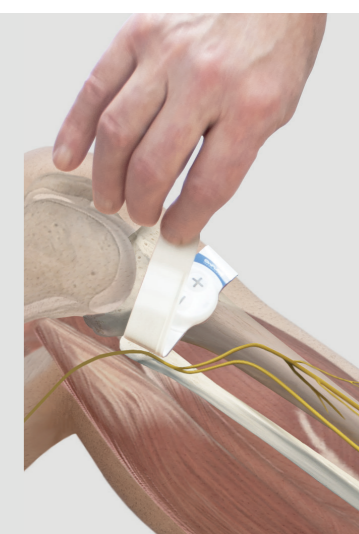

Patient instructions for the geko™ device

1. Each foil pack contains two geko™ devices. The geko™ device should always be applied to your leg(s) as instructed by your doctor. Fitting instructions can be found on reverse side
2. You will need to wear each geko™ device at home for a total of 30 hours.
3. Your geko™ will be applied right after surgery. If you find that you will need to put on a new geko™ while you are usually sleeping, you can apply a new geko™ either right before you go to bed or apply a new geko™ when you wake up in the morning.
4. If you plan to bathe or shower, turn-off and carefully remove the geko™ device. When your done, carefully dry the entire skin area where the geko™ device was located and reapply the same device for the continuation of therapy.
5. After the geko™ device runs out of battery, it can be disposed of with your household waste.
6. Check your skin carefully for any irritation before re-applying the geko™ device, if you DO experience any skin irritations please consult your fitting instructions for the alternate fitting location. If the irritations persists in new location, please remove the geko™ device and let your surgeon know.
7. Please consult your surgeon if you have any questions.

